

Adjustable Shower Chair - User Instructions


(Model: HY3520L)

The Adjustable Bath & Shower Chair by **Double Care** is intended to be used in a bath or shower to provide a stable, seated support, where a user is unable to stand for long periods. It may be used with or without the backrest and handles.

Step 1: Unpack

Contents:


- 2 x Extended legs incl. 4 x captive screws
- 5 x Screws
- 1 x Seat
- 1 x Backrest
- 2 x Backrest support tubes
- 2 x Handles

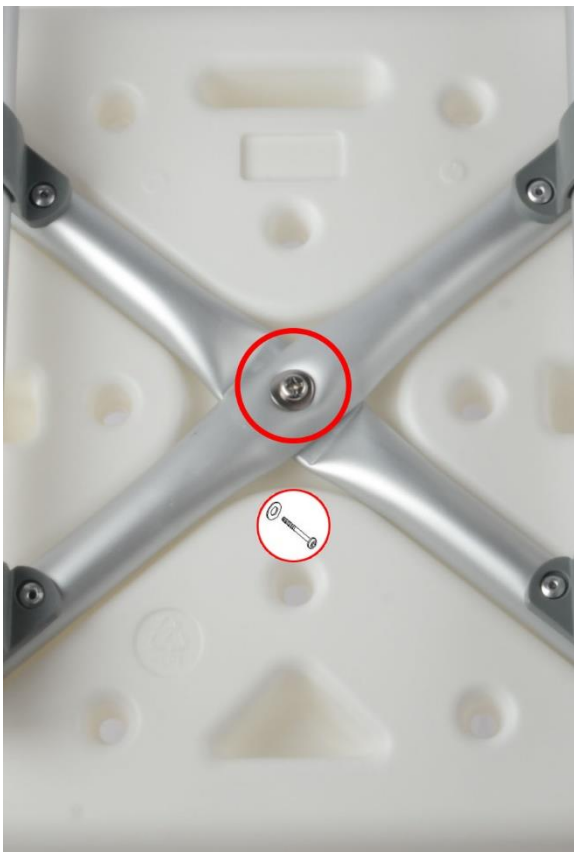

Step 3: Screw the backrest with 4 screws

Step 2: Reverse the seat first, cross 2 legs correctly
then screw the crossed legs tightly in the central of reversed seat

Step 4: Press the push buttons and card the 2 backrest support tubes into the grey casings correctly

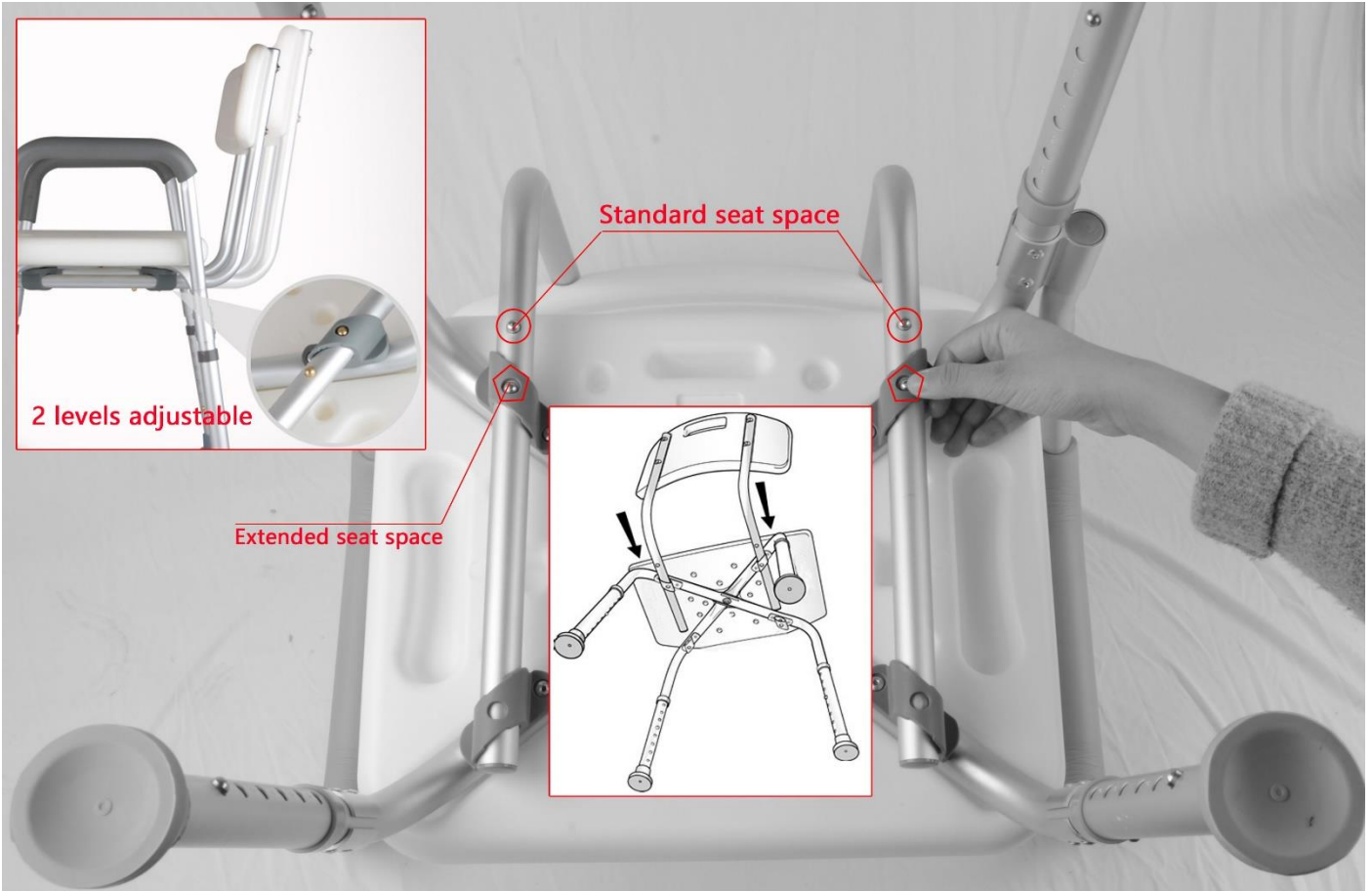

Step 5: Press Push Button to assemble the 2 handles



WARNING: Ensure that all four legs are adjusted to the same height and that each snap button is fully located.